FR

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ENG

Indications:

NPOS172 NEOPRAIR Epicondylitis strap / **NPOS173** NEOPRAIR Elbow support: Lateral epicondylitis (tennis elbow), medial epicondylitis (golfer’s elbow).
NPOS164 NEOPRAIR Bandage-style wrist support: Osteoarthritis, arthritis, tendinitis, bursitis, sprains, minor joint injuries.
NPOS167 NEOPRAIR Wraparound wrist support: Sprains, tendinitis and physical activity that puts a strain on the wrist joint. Osteoarthritis, arthritis, bursitis and post injury.
NPOS165 NEOPRAIR Wraparound wrist and thumb support: Rhizarthrosis, arthritis, tendinitis, bursitis, sprains and minor injuries.
NPOS154 NEOPRAIR Band: Mild lower back pain and when slight compression is required.

NPOS155 NEOPRAIR Lumbosacral back support: Acute and chronic low back pain, minor injuries, heat therapy and sports use with lower back pain.

Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- Only a doctor is qualified to prescribe the orthosis and determine the duration of treatment.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Treatment with the orthosis should be regularly monitored by your doctor.
- If any side effects are experienced, your doctor should immediately be informed. Your doctor should be informed if a skin condition appears or any type of sensitisation occurs.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- For the orthosis to function properly, its components (straps, Velcros, silicone pads, etc.) should be regularly checked to ensure that they are in good condition.
- Do not let children play with this device.
- Clean the product regularly and maintain proper hygiene.
- The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Do not wear the orthosis while having a bath, shower, etc.

Instructions for proper fitting:

During periods of use, follow the advice of the professional who recommended and/ or dispensed the product. To gain the highest degree of therapeutic effectiveness for the different conditions and to prolong the life of the product, it is essential to select the right size for each patient or user. As excessive compression can lead to intolerances, we recommend adjusting the compression to the optimum degree. Fit the product to the joint in question, making sure that it is properly aligned to enable correct movement of the muscles. Adjust the product to ensure a snug fit and no discomfort.

NPOS173 NEOPRAIR Elbow support: (See pictures)

- Fit the orthosis to the arm and position it correctly.
- Fasten the straps until the desired pressure is obtained.

Product maintenance - washing instructions:
Before washing the support, fasten all of the Velcros and detach the removable components (silicone pads, splints, palmar stays, etc.). Hand wash. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Do not expose the orthosis to direct heat sources such as stoves, heaters, radiators and direct sunlight.

Classification:

Class I medical device without a measuring function and supplied in a non-sterile condition. This orthosis complies with Council Directive 93/42/EEC, amended by 2007/47/EC, concerning medical devices.

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